## BM 47

<b>D</b>	<b>Blutdruckmessgerät</b> Gebrauchsanweisung .....	2-14
<b>GB</b>	<b>Blood pressure monitor</b> Instructions for use .....	15-26
<b>FR</b>	<b>Tensiométre</b> Mode d'emploi.....	27-38
<b>ES</b>	<b>Tensiómetro</b> Manual de instrucciones .....	39-51
<b>IT</b>	<b>Misuratore di pressione</b> Istruzioni per l'uso .....	52-63
<b>TR</b>	<b>Bilgisayarli tansiyon ölçer</b> Kullanım kılavuzu .....	64-74
<b>RUS</b>	<b>Прибор для измерения кровяного давления в плечевой артерии</b> Инструкция по применению .....	75-88
<b>PL</b>	<b>Ciśnieniomierz</b> Instrukcja obsługi .....	89-101
	<b>Electromagnetic Compatibility Information .....</b>	102-105

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# ENGLISH

## Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,  
Your Beurer Team

## 1. Getting to know your instrument

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure.

You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

A warning is issued for anyone suffering from cardiac arrhythmia.

The recorded values are classified and evaluated graphically.

Store these instructions for use for future reference and make them accessible to other users.


## 2. Important information

### Signs and symbols

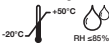The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Note Note on important information
	Follow instructions for use
	Type BF applied part
	Direct current

	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
<b>Storage</b>  -20°C to +50°C RH ≤85%	Permissible storage temperature and humidity
<b>Operating</b>  +10°C to +40°C RH ≤85%	Permissible operating temperature and humidity
	Keep dry
	Serial number
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

 **Advice on use**

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Before every measurement, relax for about five minutes.

- If you want to perform several measurements on the same person, wait five minutes between each measurement.
- Do not take a measurement within 30 minutes after eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are not a substitute for a medical examination! Discuss the measurements with your doctor, and never base any medical decisions on them (e.g. medicines and their administration)!
- Do not use the blood pressure monitor on newborns, pregnant women or patients with preeclampsia.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the device on people who have the specified upper arm measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.

- Avoid any mechanical restriction, compression or bending of the cuff line.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- The blood pressure monitor can only be operated with batteries. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty, the blood pressure monitor loses the date and time.
- To conserve the batteries, the monitor switches off automatically if no buttons are pressed for one minute.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

### **Storage and Care**

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.
  - You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
  - Never drop the device.
  - Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
  - Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- Do not press any buttons until the cuff is in position.
- If the instrument is not used for any length of time, we recommend removing the batteries.

### **Notes on handling batteries**

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.

- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.

- **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.


### **Repair and disposal**

- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.
- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.

- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.


### 3. Unit description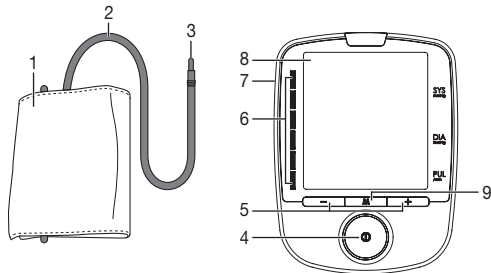

1. Cuff
2. Cuff line
3. Cuff connector
4. START/STOP button **ⓘ**
5. function buttons -/+
6. Scale for classifying the measurements
7. Connection for cuff connector
8. Display
9. Memory button **M**

### Information on the display:

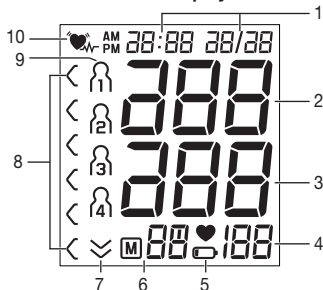
1. Time and date
2. Systolic pressure
3. Diastolic pressure
4. Calculated pulse value
5. Battery replacement symbol
6. Memory display: average value (R), morning (R<sup>m</sup>), evening (P<sup>m</sup>), memory space number
7. Release air
8. Classification of measurements
9. User memory
10. Cardiac arrhythmia symbol

## 4. Preparing the measurement

### Insert the batteries

- Open the battery compartment lid.
- Insert four 1.5 V AA (alkaline type LR6) batteries. Make sure that the batteries are inserted with the correct polarity, according to the label. Do not use rechargeable batteries.
- Close the battery compartment lid again carefully.
- With inserted batteries, the unit permanently displays the time, date and selected user memory.

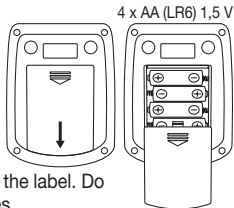If the low battery indicator  is permanently displayed, you can no longer perform any measurements and must replace the batteries. Once the batteries have been removed from the unit, the time must be set again.

Used batteries should not be disposed of in normal household waste. Dispose of them via your electronics retailer or your local recycling point. You are legally obligated to do so.


### Battery disposal

- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling

points or electronics retailers. You are legally required to dispose of the batteries.

- The codes below are printed on batteries containing harmful substances:  
Pb = Battery contains lead,  
Cd = Battery contains cadmium,  
Hg = Battery contains mercury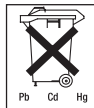

### Setting the date and time

It is essential to set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and to access them again later. The time is displayed in the 24-hour format.

To set the date and time, proceed as follows:

- Press the memory button **M** for at least three seconds.
- The year display will start to flash. Set the year with the function buttons **-/+** and confirm with the memory button **M**.
- Set the month, day, hour and minutes and confirm each setting with the memory button **M**.

## 5. Measuring blood pressure




Please ensure the unit is at room temperature before measuring.

## Attach the cuff

Place the cuff on to the bare left upper arm. The circulation of the arm must not be hindered by tight clothing or similar.

The cuff must be placed on the upper arm so that the bottom edge is positioned 2–3 cm above the elbow and over the artery. The line points to the centre of the palm.

Now tighten the free end of the cuff, but make sure that it is not too tight around the arm and close the hook-and-loop fastener. The cuff should be fastened so that two fingers can fit under the cuff.

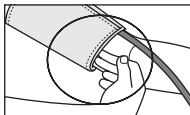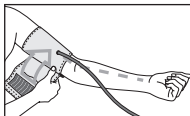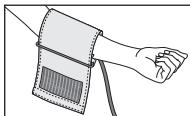Now insert the cuff line into the connection for the cuff connector.



**Caution:** The unit may only be operated with the original cuff. The cuff is suitable for an arm circumference of 22 to 35 cm.

A larger cuff for upper arm circumferences from 30 to 42 cm is available from retailers or the service address under order no. 162.795.

⚠ Please ensure that the cuff is not bent at any point and that the reinforced end lies flat against your arm.

## Adopt the correct posture

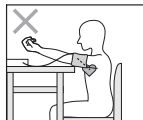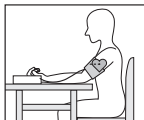- Before every measurement, relax for about five minutes. Otherwise deviations can occur.
- You can perform the measurement while sitting or lying. Make sure that the cuff is at heart level.
- To carry out a blood pressure measurement, make sure you are sitting comfortably with your arms and back lean-

ing on something. Do not cross your legs. Place your feet flat on the ground.

- To avoid falsifying the measuring result, it is important to remain still during the measurement and not to speak!

### Selecting memory

Press the - or + function button. Select the desired user memory (M<sub>1</sub> ... M<sub>4</sub>) by pressing the -/+ function button. You have 4 memories with 30 spaces each to store the measurements of 4 different people separately. Confirm your selection with the START/STOP button ①.

### Performing the blood pressure measurement

- As described before, attach the cuff and adopt the posture in which you want to perform the measurement.
- Start the blood pressure monitor with the START/STOP button ①. After the full-screen display, the most recently stored measurement appears. If there is no measurement in the memory, the unit displays the value "0".
- The cuff automatically inflates. The cuff's air pressure is slowly released. If the device already recognise a tendency for high blood pressure, it will reinflate and increase the cuff pressure again. As soon as a pulse is found, the pulse symbol ♥ is displayed.
- Systolic pressure, diastolic pressure and pulse readings are displayed.

- Measuring can be cancelled at any time by pressing the START/STOP button ①.
- Er\_ appears if the measurement has not been performed properly. Take note of the chapter on error messages/ trouble-shooting in these instructions for use and repeat the measurement.
- The measurement is automatically stored.
- To switch off, press the START/STOP button ①. If you forget to turn off the unit, it will switch off automatically after approx. 1 minute.

Wait at least 5 minutes before taking another measurement.

## 6. Evaluating results

### Cardiac arrhythmia:

This unit can identify potential disruption of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol ♥<sub>ir</sub>. This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heartbeat. The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor. If the symbol ♥<sub>ir</sub> is shown on the display after the measurement has been taken, it should be repeated.

Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

### **Classification of measurements:**

The measurements can be classified and evaluated in accordance with the following table.

However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice.

Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The bar chart on the display and the scale on the unit show which category the recorded blood pressure values fall into.

If the values of systole and diastole fall into two different categories (e.g. systole in the High normal category and diastole in the Normal category), the graphical classification on the device always shows the higher category; for the example given this would be High normal.

<b>Blood pressure value category</b>	<b>Systole (in mmHg)</b>	<b>Diastole (in mmHg)</b>	<b>Action</b>
Setting 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Setting 2: moderate hypertension	160–179	100–109	seek medical attention
Setting 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor
Normal	120–129	80–84	self-monitoring
Optimal	< 120	< 80	self-monitoring

Source: WHO, 1999 (World Health Organization)

## **7. Saving, displaying and deleting measurements**

- The results of every successful measurement are stored together with the date and time. If there are more than 30 measurements, the oldest measurements are lost.
- Press the - or + function button. Select the desired user memory (M<sub>1</sub> ... M<sub>4</sub>) by pressing the -/+ function button. Confirm your selection with the START/STOP button .

- If you press the memory button **M**, the average value  $\bar{R}$  of all the stored measured values in the user memory will be displayed. If you press the memory button **M** again, the average value of the morning measurements for the last 7 days will be displayed. (morning: 5.00 – 9.00, display  $\bar{R}^m$ ). If you press the memory button **M** again, the average value of the evening measurements for the last 7 days will be displayed. (evening: 18.00 – 20.00, display  $\bar{R}^e$ ). If you continue to press the memory button **M**, the most recent individual measured values are displayed in turn with the date and time.
- You can delete the memory by pressing and holding the memory button **M** for 3 seconds.
- To switch off, press the START/STOP button **ⓘ**.
- If you forget to switch off the unit, it will switch off automatically after 1 minute.

## 8. Error message/trouble shooting

In the event of errors, the error message  $E_r\_$  appears on the display.

Error messages may appear if:

- systolic or diastolic pressure could not be measured ( $E_r 1$  or  $E_r 2$  appears on the display)
- systolic or diastolic pressure was outside the measurement range (**Hi** or **Lo** appears on the display)

- the cuff is fastened too tightly or loosely ( $E_r 3$  or  $E_r 4$  appears on the display)
- the pump pressure is higher than 300 mmHg ( $E_r 5$  appears on the display)
- pumping up takes longer than 160 seconds ( $E_r 6$  appears on the display)
- there is a system or device error ( $E_r A$ ,  $E_r D$ ,  $E_r 7$  or  $E_r B$  appears on the display)
- the batteries are almost empty .

In such cases, repeat the measurement. Ensure that you do not move or speak.

If necessary, reinsert or replace the batteries.

### **ⓘ** Technical alarm – description

Should the recorded blood pressure (systolic or diastolic) lie outside the limits specified in the section “Technical specifications”, the technical alarm will appear on the display indicating either “**Hi**” or “**Lo**”. In such cases, you should seek medical assistance and check the accuracy of your procedure.

The limit values for the technical alarm are factory set and cannot be adjusted or deactivated. These alarm limit values are accorded second priority under the standard IEC 60601-1-8.

The technical alarm is a non-locking alarm and must not be reset. The signal shown on the display will disappear automatically after about 8 seconds.

## 9. Cleaning and storing the unit

- Clean your blood pressure device carefully with a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances should you hold the unit under water, as this can cause liquid to enter and damage the unit.
- If you store the unit, no heavy objects should be placed on top of it. Remove the batteries. The cuff line should not be bent sharply.

## 10. Technical specifications

Model no.	BM 47
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0-300 mmHg, systolic 60-260 mmHg, diastolic 40-199 mmHg, Pulse 40-180 beats/minute

Display accuracy	Systolic $\pm 3$ mmHg, diastolic $\pm 3$ mmHg, pulse $\pm 5\%$ of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg/diastolic 8 mmHg
Memory	4 x 30 memory spaces
Dimensions	L 134 mm x W 102 mm x H 52,5 mm
Weight	Approximately 280g (without batteries and cuff)
Cuff size	22 to 35 cm
Permissible operating conditions	+10 °C to +40 °C, $\leq 85\%$ relative air humidity (non-condensing)
Permissible storage conditions	-20 °C to +50 °C, $\leq 85\%$ relative air humidity, 800-1050 hPa ambient pressure
Power supply	4 x 1,5V  AA batteries
Battery life	For approx. 500 measurements, depending on the blood pressure level and/or pump pressure
Accessories	Instruction for use, 4 x 1.5V AA batteries, storage pouch

---

Classification	Internal supply, IPX0, no AP or APG, continuous operation, type BF applied part
----------------	---

---

Technical information is subject to change without notification to allow for updates.

- This unit is in line with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.
- This device is in line with the EU Medical Devices Directive 93/42/EEC, the „Medizinproduktegesetz“ (German Medical Devices Act) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2–30: Particular requirements for the safety and essential performance of automated non-invasive blood pressure monitors).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical pur-

poses, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

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Table 1

For all ME EQUIPMENT and ME SYSTEMS

## *Guidance and manufacture's declaration – electromagnetic emissions*

The BM 47 is intended for use in the electromagnetic environment specified below.  
The customer or the user of the BM 47 should assure that it is used in such an environment.

<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment – guidance</b>
RF emissions CISPR 11	Group 1	The BM 47 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The BM 47 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	not applicable	

**Table 2****For all ME EQUIPMENT and ME SYSTEMS***Guidance and manufacturer's declaration – electromagnetic immunity*

The BM 47 is intended for use in the electromagnetic environment specified below. The customer or the user of the BM 47 should assure that it is used in such an environment.

<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE:  $U_T$  is the a.c. mains voltage prior to application of the test level.

**Table 3****For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING***Guidance and manufacturer's declaration – electromagnetic immunity*

The BM 47 is intended for use in the electromagnetic environment specified below. The customer or the user of the BM 47 should assure that it is used in such an environment.

<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
----------------------	-----------------------------	-------------------------	---

Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BM 47, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance:</b></p> $d = 1.2\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3\sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range<sup>b</sup>. Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p><sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BM 47 is used exceeds the applicable RF compliance level above, the BM 47 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the BM 47.</p> <p><sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

**Table 4****For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING***Recommended separation distances between portable and mobile RF communications equipment and the BM 47*

The BM 47 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BM 47 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BM 47 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz not applicable	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2.3\sqrt{P}$
0,01	-	0,12	0,23
0,1	-	0,38	0,73
1	-	1,2	2,3
10	-	3,8	7,3
100	-	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

---

BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)  
[www.beurer.de](http://www.beurer.de)