BC 40

(D)	Blutdruckmessgerät Gebrauchsanweisung	2-14
(GB)	Blood pressure monitor Instructions for use	15-26
(F)	Tensiométre Mode d'emploi	27-38
(E)	Tensiómetro Manual de instrucciones	39-51
(I)	Misuratore di pressione Istruzioni per l'uso	52-63
(TR)	Bilgisayarli tansiyon ölçer Kullanım kılavuzu	64-74
(RUS)	Прибор для измерения кровяного давления на запястье Инструкция по применению	75-88
(PL)	Ciśnieniomierz Instrukcja obsługi	89-99
	Electromagnetic Compatibility Information	100-105

Inhalt

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ENGLISH

Contents

1. Getting to know your device	15	6. Saving, displaying and deleting measurements	23
2. Important notes	16	7. Error messages/troubleshooting	24
3. Unit description	19	8. Cleaning and maintenance.....	24
4. Preparing the measurement	19	9. Technical specifications	25
5. Measuring blood pressure.....	21		

Dear customer,

Thank you for choosing one of our products. Our brand stands for well-engineered, quality made products; driven by innovation and safety. Our product range encompasses a wealth of products for health and well-being in the heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air sectors.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

1. Getting to know your device

The wrist blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure.

This allows you to quickly and easily measure your blood pressure and to display the last recorded measurement. A warning is issued for anyone suffering from cardiac arrhythmia.

The recorded values are classified and evaluated graphically. Store these instructions for use for future reference and make them accessible to other users.

2. Important notes

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

	Attention
	Note Note on important information
	Observe the instructions for use
	Application part, type BF
	Direct current
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer

Storage 	Permissible storage temperature and humidity
Operating 	Permissible operating temperature and humidity
IP22	Protection against harmful ingress of water and dust. Protected against foreign objects equal to or greater than 12.5 mm in diameter and against drops of water falling at up to 15° from vertical
SN	Serial number
CE 0483	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

Notes on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Before every measurement, relax for about five minutes.
- If you want to perform several measurements on the same person, wait five minutes between each measurement.

- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are no substitute for a medical examination. Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).
- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- In the case of restricted circulation on the arm as a result of chronic or acute vascular diseases (including vascular constriction), the accuracy of the wrist measurement is limited. In this case you should avoid using an upper arm blood pressure monitor.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the unit on people who have the specified wrist measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or intravascular therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- The blood pressure monitor can only be operated with batteries.
- To conserve the batteries, the blood pressure monitor switches off automatically if you do not press any buttons for 3 minutes.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

Instructions for storage and maintenance

- The blood pressure monitor is made from precision and electronic components. The accuracy of the measurements and service life of the device depend on its careful handling:
 - Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
 - Do not drop the device.
 - Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
 - Only use the cuff included with the delivery or original replacement parts. Otherwise incorrect measurements will be recorded.
- Do not press the buttons before the cuff is placed on the arm.
- We recommend that the batteries be removed if the device will not be used for a prolonged period of time.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.

-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

Instructions for repairs and disposal

- Batteries do not belong in household waste. Please dispose of empty batteries at the collection points intended for this purpose.
- Do not open the device. Failure to comply will invalidate the warranty.
- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.

- Repairs must only be carried out by Customer Services or authorised suppliers.

Before making a claim, please check the batteries first and replace them if necessary.

- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

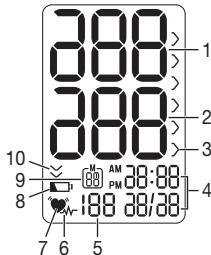
3. Unit description

1. Display
2. Battery compartment lid
3. Wrist cuff
4. START/STOP button **ⓘ**
5. Memory button **M**

Information on the display:

1. Systolic pressure
2. Diastolic pressure
3. Classification of measurements
4. Time and date
5. Calculated pulse value
6. Cardiac arrhythmia symbol 
7. Pulse symbol 
8. Battery replacement symbol 
9. Memory space number/
memory display average value (A), morning (AM), evening (PM)
10. Release air (arrow)

4. Preparing the measurement

Insert the batteries

- Remove the battery compartment lid on the right side of the device.
- Insert two 1.5 V micro (alkaline type LR03) batteries. Make sure that the batteries are inserted the correct way round. Do not use rechargeable batteries.
- Close the battery compartment lid again carefully.

If the battery replacement symbol  is displayed, you can no longer perform any measurements and must replace all batteries.

Battery disposal

- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury

Setting the hour format, date and time

This menu allows you to set the following functions, one after another.

It is essential to set the date and time. Otherwise, you will not be able to save your measurements correctly with a date and time and access them again later.

-  If you press and hold the memory button **M** you can set the values more quickly.

To access the settings mode, either reinsert the batteries or press and hold the START/STOP button  for 5 seconds when the device is switched off.

Hour format

- Select the desired hour format using the memory button **M** and confirm with the START/STOP button .

Date

The year flashes on the display.

- Select the desired year using the memory button **M** and confirm with the START/STOP button .

The month flashes on the display.

- Select the desired month using the memory button **M** and confirm with the START/STOP button .

The day flashes on the display.

- Select the desired day using the memory button **M** and confirm with the START/STOP button .

-  If the hour format is set as 12h, the day/month display is reversed.

Time

The hours flash on the display.

- Select the desired hour using the memory button **M** and confirm with the START/STOP button **I**.

The minutes flash on the display.

- Select the desired minute using the memory button **M** and confirm with the START/STOP button **I**.

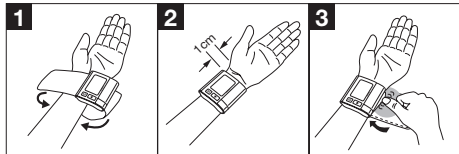
Once all the data has been set, the device switches off automatically.

5. Measuring blood pressure

Please ensure the unit is at room temperature before measuring.

Attaching the cuff

- Expose your left wrist. Ensure that the circulation of the arm is not hindered by tight clothing or similar. Place the cuff on the inside of your wrist.

- Fasten the cuff with the hook and loop fastener so that the upper edge of the monitor is positioned approx. 1 cm below the ball of your thumb.
- The cuff must be fitted tightly around the wrist, but must not constrict.

Important: The unit may only be operated with the original cuff.

Adopt the correct posture

- Before every measurement, relax for about five minutes. Otherwise deviations can occur.
- You can take the measurement while sitting or lying. To take your blood pressure, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground. Make sure to rest your arm and move it. Always make sure that the cuff is at heart level. Otherwise significant deviations can occur. Relax your arm and the palm of your hand.
- To avoid falsifying the measurement, it is important to remain still during the measurement and not to speak.
- Let your hand and wrist relax or form a loose fist. Keep your hand and wrist in the same position throughout the measurement process.

Performing the blood pressure measurement

- Start the blood pressure monitor with the button .
 - All displays will illuminate briefly.
 - The blood pressure monitor will begin the measurement automatically after 3 seconds. The measurement is taken during the inflation phase.
 - As soon as a pulse is found, the pulse symbol  is displayed.
 - The systolic pressure, diastolic pressure and pulse rate measurements are displayed.
-  Measuring can be cancelled at any time by pressing the  button.
- **EE_** appears if the measurement has not been performed properly.
Observe the chapter on error messages/troubleshooting in these instructions for use and repeat the measurement.
 - The measurement is automatically stored.

Evaluating results

Cardiac arrhythmia:

This device can identify potential disruptions of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol .

This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heart-

beat. The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor.

If the symbol  is shown on the display after the measurement has been taken, repeat the measurement. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

Classification of measurements:

The measurements can be classified and evaluated in accordance with the following table.

However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice.

Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the values of systole and diastole fall into two different categories (e.g. systole in the 'High normal' category

and diastole in the 'Normal' category), the graphical classification on the device always shows the higher category; for the example given this would be 'High normal'.

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Setting 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Setting 2: moderate hypertension	160–179	100–109	seek medical attention
Setting 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor
Normal	120–129	80–84	self-monitoring
Optimal	< 120	< 80	self-monitoring

Source: WHO, 1999 (World Health Organization)

6. Saving, displaying and deleting measurements

The results of every successful measurement are stored together with the date and time. If there are more than 60 measurements, the oldest measurements are lost.

- To access the memory recall mode, press the memory button **M**.

Your last measurement will appear on the display.

Displaying average values

- Press the memory button **M**. **A** flashes on the display. The average value of all saved measurements is displayed.
- Press the memory button **M**. **AM** flashes on the display. The average value of the morning measurements for the last 7 days is displayed (morning: 5.00 a.m. – 9.00 a.m.).
- Press the memory button **M**. **PM** flashes on the display. The average value of the evening measurements for the last 7 days is displayed (evening: 6.00 p.m. – 8.00 p.m.).

Displaying individual measurements

- When the memory button **M** is pressed again, the last individual measurement is displayed.
- When the memory button **M** is pressed again, you can view your individual measurements.
- To switch the device off again, press the START/STOP button **I**.

- i** You can exit the menu at any time by pressing the START/STOP button **I**.

Deleting measurements

Deleting individual measurements

- Start individual measurement access.
- Press and hold the memory button **M** for approx. 5 seconds. The relevant measurement is deleted.
- The display starts to flash, then an empty display with empty user memory frame is displayed. Press the memory button **M** to delete additional values or press the START/STOP button **I** to switch off the device.

Deleting all measurements

- Start access to average value of all saved measurements by pressing the memory button **M** twice.
- Press and hold the memory button **M** for approx. 5 seconds. All measurements are deleted.

- The display starts to flash, then an empty display with empty user memory frame is displayed. Press the memory button **M** to delete additional values or press the START/STOP button **I** to switch off the device.

7. Error messages/troubleshooting

In the event of errors, the error message **EE_** appears on the display.

Error messages may appear if:

- you move or speak during the measurement (**EE 1** appears on the display)
- the cuff is placed on the arm too tightly or loosely or if the cuff is damaged (**EE 2** appears on the display)
- the pump pressure is higher than 290 mmHg (**EE B** appears on the display)
- the batteries are almost worn out (**Lo** and ).

In such cases, repeat the measurement. If necessary, reinsert or replace the batteries.

8. Cleaning and maintenance

- Clean your blood pressure monitor carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances should you hold the unit under water, as this can cause liquid to enter and damage the unit.
- Never place any heavy objects on the device.

9. Technical specifications

Model no.	BC 40
Measurement method	Oscillometric, non-invasive blood pressure measurement on the wrist
Measurement range	Cuff pressure 0 – 290 mmHg, Systolic 50 – 250 mmHg, Diastolic 40 – 180 mmHg, Pulse 40 – 180 beats/minute
Display accuracy	Systolic ± 3 mmHg, diastolic ± 3 mmHg, pulse ± 5 % of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg/diastolic 8 mmHg
Memory	60 memory spaces
Dimensions	L 76 mm x W 70 mm x H 23 mm
Weight	Approximately 105 g (without batteries)
Cuff size	12.5 to 21.5 cm
Permissible operating conditions	+ 10 °C to + 40 °C, ≤ 85 % relative humidity (non-condensing)
Permissible storage conditions	- 20 °C to + 55 °C, ≤ 85 % relative humidity, 800-1060 hPa ambient pressure
Power supply	2 x 1.5V  AAA batteries

Battery life	For approx. 300 measurements, depending on levels of blood pressure and pump pressure
Accessories	Instructions for use, 2 x 1.5V AAA batteries, storage box
Classification	Internal supply, continuous operation, application part type BF, IPX0, no AP or APG

Technical information is subject to change without notification to allow for updates.

- This device is in line with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Services address or found at the end of the instructions for use.
- This device corresponds to the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktegesetz) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC80601-2-30 (Medical electrical equipment – Part 2 – 30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).

- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

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Electromagnetic Compatibility Information

Table 1

<i>Guidance and manufacturer's declaration – electromagnetic emission</i>		
The BC40 Wrist Digital Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of BC40 Wrist Digital Blood Pressure Monitor should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BC40 Wrist Digital Blood Pressure Monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The BC40 Wrist Digital Blood Pressure Monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Table 2

<i>Guidance and manufacturer's declaration – electromagnetic immunity</i>			
The BC40 Wrist Digital Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BC40 Wrist Digital Blood Pressure Monitor should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	± 1 kV differential mode ± 2 kV common mode	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the BC40 Wrist Digital Blood Pressure Monitor requires continued operation during power mains interruptions, it is recommended that the BC40 Wrist Digital Blood Pressure Monitor be powered from an uninterruptible power supply or a battery.

Guidance and manufacturer's declaration – electromagnetic immunity

The BC40 Wrist Digital Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BC40 Wrist Digital Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	< 5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles < 5 % UT (>95 % dip in UT) for 5 sec	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE 1 U_T is the a. c. mains voltage prior to application of the test level.

Table 3
Guidance and manufacturer's declaration – electromagnetic immunity

The BC40 Wrist Digital Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BC40 Wrist Digital Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3V _{rms} 150 kHz to 80 MHz	Not applicable	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BC40 Wrist Digital Blood Pressure Monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{V_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2,5 GHz	3V/m	$d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).b</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,a should be less than the compliance level in each frequency range.b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.</p>			

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BC40 Wrist Digital Blood Pressure Monitor is used exceeds the applicable RF compliance level above, the BC40 Wrist Digital Blood Pressure Monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the BC40 Wrist Digital Blood Pressure Monitor.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Table 4

Recommended separation distances between portable and mobile RF communications equipment and the BC40 Wrist Digital Blood Pressure Monitor

The BC40 Wrist Digital Blood Pressure Monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BC40 Wrist Digital Blood Pressure Monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BC40 Wrist Digital Blood Pressure Monitor as recommended below, according to the maximum output power of the communications equipment

Rated maximum output of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0,01	-	0,12	0,23
0,1	-	0,38	0,73
1	-	1,2	2,3
10	-	3,8	7,3
100	-	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.