DEUTSCH

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ENGLISH

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Please read these instructions carefully before using the device.

LOT	LOT number
Expiry date	Expiry date
Manufacturer	Manufacturer
3V CR2032	1 x CR2032 battery
Important!	Read the instructions for use.
Application part, type BF	Application part, type BF

1. Getting to know your device

1.1 What is the EMS pad?

The innovative EMS Neck Pad is a self-adhesive EMS device that has been specially developed for relaxing the neck and shoulder muscles. The device can be applied with accuracy thanks to the use of EMS technology. Made from medical-quality silicone rubber, the elegantly designed EMS pad is extremely slim and flexible and adapts perfectly to the area to be treated. The high-tech circuit provides energy-efficient treatments of 20 minutes each (up to 20 treatments per battery).

1.2 What is EMS and what can it do?

Electrical muscle stimulation (EMS) is a widespread and generally recognised method and has been used in sports medicine and rehabilitation for years. In sports and fitness, EMS is used to complement conventional muscle training, to increase the performance of muscle groups and to adjust physical proportions to achieve the desired aesthetic results. There are two different types of EMS application: One is for targeted strengthening of the muscles (activating application), and the other is to achieve a relaxing, restful effect (relaxing application).

The activating application involves:

- Muscle training to increase endurance and/or
- Muscle training to support the strengthening of specific muscles or muscle groups, and to achieve the desired changes to physical proportions.

The relaxing application involves:

- Muscle relaxation for easing muscle tension.
- Improving symptoms of muscular fatigue.
- Acceleration of muscle regeneration after high muscle performance (e.g. after a marathon).

EMS devices work by passing electrical currents over the skin. The gel film is used as a transfer medium and is subject to natural wear and tear. The gel film must be replaced if it stops providing sufficient contact, as this will prevent the EMS pad

from sticking to the skin. If it is not replaced, the partially increased current density could irritate the skin.

1.3 Included in the delivery

- 1 x EMS pad
- 1 x self-adhesive gel film set
- 1 x 3V CR2032 battery

Replacement items

- 1 x self-adhesive gel film set and 1 x battery, item no.: 647.17
- When necessary, please contact Customer Services at the address provided in good time.

IMPORTANT!

The EMS pad must only be used

- on people.
- For the intended purpose and as specified in these instructions for use. Any improper use can be dangerous.
- For external use.
- Use only with the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty.

2. Important notes

The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or are suffering from an illness.

Instructions for application

WARNING!

To avoid damage to health, we strongly advise against using the EMS pad in the following situations:

- If you have implanted electrical devices (e.g. a pacemaker).
- If you have metal implants.
- If you use an insulin pump.
- If you have a high temperature (e.g. > 39 °C).
- If you have a known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system.
- If you suffer from a seizure disorder (e.g. epilepsy).
- If you are pregnant.
- If you have cancer.
- After an operation, if strong muscle contractions could affect the healing process.
- If you suffer from metabolic arthritis.
- The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), especially not on the two large pectorals. This can increase the risk of ventricular fibrillation and induce cardiac arrest.
- On the skeletal skull structure, or around the mouth, throat or larynx.
- In the neck area/carotid artery area/above the neck.
- In the genital area.

- On acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin - whether painful or not, reddened skin, rashes, e.g. allergies, burns, bruises, swellings, both open and healing wounds, and post-operative scars where the healing process could be affected).
- In humid environments (e.g. in the bathroom) or when bathing or showering.
- Do not use after consuming alcohol.
- Do not use if connected to a high-frequency surgical device.
- This unit is not intended for use by children or persons with physical, sensory (e.g. insensitivity to pain) or mental impairments or those who do not know how to use it due to a lack of experience or knowledge, unless they are supervised by a person responsible for their safety or have received instructions from them on how to use the unit.

Before using the device, consult your doctor if any of the following applies to you:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant disorders.
- Skin diseases or open wounds.
- Unexplained chronic pain in any part of the body.
- Diabetes.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- If you are receiving medical treatment.
- If you are already having stimulation treatment for existing conditions.
- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site.

PRECAUTIONS:

- Hold the device away from sources of heat and do not use it in close proximity (approx. 1 m) to shortwave or microwave devices (e.g. mobile phones), as doing so can result in unpleasant current peaks.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity. Never immerse the device in water or other liquids.
- The device is suitable for self-treatment.
- For hygienic reasons, the device may only be used on one person.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Switch off the EMS pad before attaching or removing it.
- Do not adapt the EMS pad in any way (e.g. by cutting or trimming it). This will produce a higher current density and can be dangerous.
- Do not use whilst asleep, driving a vehicle or operating machinery.
- Do not use whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- Ensure that no metallic objects (e.g. belt buckles or necklaces) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated (e.g. a navel piercing), these must be removed before using the device. Failure to do so could result in spot burns.
- Keep the device away from children.
- Pre-treating the skin with greasy creams or lotions is not recommended.

- Do not use adhesive tapes, bandages or any aid other than the designated gel film to apply the EMS pad to your skin. Otherwise the unequal distribution of the gel film could lead to skin lesions.
- The device should be used under medical supervision where possible.
- Stimulation on the same area for a long period of time could irritate the skin. Please consult a dermatologist if skin irritation persists.
- During the initial few minutes, use the device while sitting or lying down to minimise the risk of accidental injuries as a consequence of isolated cases of vagal responses (feeling of faintness). If you feel faint, switch off the device immediately, lie down and support the legs in an elevated position (approx. 5 – 10 min).

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.
- Remove the battery from the item if it is empty or you do not intend to use the item for a prolonged period of time. This prevents damage which could occur due to leakage.
- If required, clean the battery and device contacts before inserting. Risk of overheating!

Damage

- If the EMS pad is damaged or you have any doubts, do not use it and contact your retailer or Customer Services at the address provided.
- Switch the device off immediately if it is faulty or not working properly.
- Do not attempt to open and/or repair the EMS pad yourself. Repairs may only be carried out by Customer Services or authorised suppliers. Failure to comply will result in voiding of the warranty.
- The manufacturer is not liable for damage resulting from improper or careless use.

Cleaning

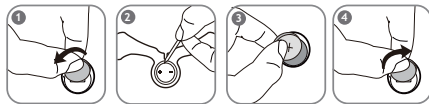
- Remove the batteries from the device before cleaning.
- Clean the device after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.
- Ensure that no water gets into the device. If this happens, only use the device again once it has fully dried out.
- Do not use any chemical or abrasive cleaning agents.

Storage

- Re-apply the protective film to the gel film after use.
- If you do not plan to use the device for a long period of time, we recommend that you store it in the original packaging in a dry environment, ensuring it is not weighed down by additional items.

3. Initial use

3.1 Inserting the battery

1. Turn the battery compartment lid anti-clockwise using a coin.
2. Use a pointed object to loosen the lid.
3. Place the battery in the battery compartment so that the positive terminal (+) faces upwards.
4. Close the lid by turning it clockwise.

3.2 Attaching the gel film

1. Carefully remove the blue protective film.

2. Carefully apply the gel film onto the EMS pad and then carefully peel away the protective film. Ensure that the edge of the gel film does not protrude over the EMS pad.

Gel film that has been applied slightly askew will not affect the function.

WARNING!

- Make sure that you attach the gel film to the device on the side where the blue protective film was, otherwise the skin could become irritated.

3.3 Usage

Stick the EMS pad onto the part of the body to be treated (see image).

1. Press and hold down the "+" button until you hear a long beep.

2. Press the "+" button again to start the Level 1 stimulation. Keep pressing the "+" button until you reach the desired intensity.

3. Press the "-" button to reduce the intensity.

To turn off the EMS pad, press and hold down the "-" button until you hear a long beep. The device switches off automatically after 20 minutes.

Re-apply the protective film to the gel film after use.

For optimal results we recommend a 4-week training plan. Please note the following points:

1. 1-2 days rest.
2. Increase the intensity level on a daily basis.
3. A higher intensity level can lead to better results. However, the stimulation should never be uncomfortable.
4. Complete the full 20-minute program each time you use the device.

Wait at least 30 minutes after the stimulation programme is complete before using the device again.

Important!

- The device will only work if there is sufficient contact with the skin. Stimulation pulses will not be transmitted if there is insufficient contact.
- Switch off the EMS pad before removing it.
- Replace the battery if it is low (insufficient stimulation, no beeping noises).
- For optimum stimulation we recommend that the gel film is replaced when batteries are replaced.
- Do not use the EMS pad on the same area more than once per day.
- In the event of continuous beeping, shut down the device. Clean the gel film and try to restart the program. If the continuous beeping persists, replace the gel film.
- If the adhesive capability of the gel film decreases, replace immediately. Do not use the EMS pad again until the gel film has been replaced. Otherwise the unequal distribution of the gel film could lead to skin lesions.
- In the event of unpleasant current peaks, switch off the EMS pad, check the electrodes and then re-apply.
- The "Smart Contact Detection" automatically triggers an audible alarm if 2 electrodes come into contact with each other.

4. Program

Cycle	1	2	3
Pulse width	400 μ s	400 μ s	400 μ s
Pulse frequency	4 Hz	6 Hz	50 Hz
On time	60 s	20 s	17 s/3*
Off time	1.5 s	1.5 s	1.5/10 s**

On time (s) = switch-on time cycle in seconds (contraction).
Off time (s) = switch-off time cycle in seconds (relaxation).

* There are 3 individual rounds in cycle 3.

** The off-time is 10 seconds for every 3rd round in cycle 3.

Note: The product components do not require any maintenance.

5. Disposal

The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronic retailers. You are legally required to dispose of the batteries. The codes below are printed on batteries containing harmful substances: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

6. Technical specifications

Type	EM 20 EMS Neck Pad
Output wave form	Biphasic rectangular pulse
Pulse length	400 μ s \pm 10%
Pulse frequency	4-6-50 Hz \pm 10%
Output voltage	max. 70 Vpp \pm 10% (1000 Ohm)
Output current	max. 70 mApp \pm 10% (1000 Ohm)
Voltage supply	3V CR2032 battery
Treatment time	20 minutes, the device then switches off automatically
Intensity	Adjustable from 0 to 15

Note:

If the device is not used according to the instructions specified, perfect functioning cannot be guaranteed!

We reserve the right to make technical changes to improve and develop the product.

7. What if there are problems?

Problem	Pulse width
Device will not switch on.	<ul style="list-style-type: none"> - Press and hold down the "+" button again for 3 seconds. - Check that the battery is inserted correctly and try to switch on the device again. - The battery could be flat. Replace the battery and try to switch on the device again.
The gel film no longer adheres to the skin.	<ul style="list-style-type: none"> - Replace the gel film.
The device beeps unusually during use.	<ul style="list-style-type: none"> - Press and hold down the "-" button for 3 seconds to switch off the device. Then press and hold down the "+" button for 2 seconds to switch on the device again. - The batteries could be low and must be replaced.
You are unable to feel any stimulation.	<ul style="list-style-type: none"> - Press and hold down the "+" button again for 3 seconds. - Ensure that the gel film does not protrude from the device and try again. - Gradually increase the intensity level.
The skin at the area of treatment is red.	<ul style="list-style-type: none"> - Stop treating this area immediately. Wait until the skin has returned to its normal colour. Please consult a dermatologist if skin irritation persists.
The intensity suddenly begins to weaken.	<ul style="list-style-type: none"> - Replace the battery and try again.

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