beurer

MG 295 HD-3D

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- D Shiatsu-Sitzauflage**
Gebrauchsanweisung..... 2
- GB Shiatsu seat cover**
Instructions for use 9
- 中 按摩垫**
使用说明16

Inhalt

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Contents

1. About Shiatsu massage	9	6. Start-up	13
2. Getting to know your appliance	10	7. Operation.....	13
3. Proper use	10	8. Care and storage.....	14
4. Safety information	10	9. In the event of problems	15
5. Appliance description	12		

Items included in the package

- Shiatsu seat cover
- These operating instructions

WARNING

- The unit is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

WARNING Warning of risks of injury or health hazards.

CAUTION Safety information about possible damage to appliance/accessories.

Note Important information.

The device has double protective insulation and therefore complies with protection class 2.

Only use indoors.

1. About Shiatsu massage

Shiatsu is a form of body massage which was originally developed in Japan around 100 years ago. It is based on ideas anchored in traditional Chinese medicine, in particular the meridian system, i.e. the paths in the human body along which energy is conveyed.

The aim of any Shiatsu massage is to promote the physical, emotional and mental wellbeing of the person being treated. To do this, energy blocks and bottlenecks in the meridians need to be dissolved and the self-regulating ability of the body needs to be stimulated.

Shiatsu therapists achieve this by exercising pressure in flowing movements along the meridians. Although Shiatsu can be literally translated as “finger pressure”, practitioners also use the heels of their hands, their elbows and knees as well as their fingers. In the process, the therapist will use his or her full body weight in order to generate the required pressure.

2. Getting to know your appliance

The massage heads in your Shiatsu massager imitate the pressure and kneading movements of a Shiatsu massage. This device with its moving massage heads provides a powerful and intense deep massage for the whole back. You have to place the Shiatsu seat cover on a suitable seat pad for this purpose.

The device is only for **private use** and is not intended for medical or commercial purposes.

3. Proper use

This device is solely intended for massaging the backs of human beings. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you.

WARNING

Use of the massage device is prohibited in the following cases:

- Not with changes caused by disease or injury in the back region (e.g. disc problems, open wounds),
- not during pregnancy,
- never while you sleep,
- not in the car,
- not on animals,
- not during activities during which an unexpected reaction may be dangerous,
- not while ingesting substances that reduce reaction times (e.g. painkiller medications, alcohol),
- never longer than 15 minutes (risk of overheating) and allow it to cool off for at least 15 minutes before using it again.

Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

The device features a switch-on heat function in the massage heads and in the seat. Persons who do not feel heat must be careful when using this function.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation on the upper body,
- you are fitted with a cardiac pacemaker, implant or other aid,
- Thromboses,
- Diabetes,
- in case of pains of unclarified origin.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

4. Safety information

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

WARNING

Keep children away from the packaging materials. Risk of suffocation.

WARNING: Electric shock

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- With the mains voltage specified on the device only,
- Do not use the device if damage to the device itself or any of its accessories is evident.
- Never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the plug from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Do not pinch, kink or twist the cables. Do not stick in any pins, needles or sharp objects.

Make sure that the massage device, the switch, the power supply unit and the cable do not come into contact with water, steam or other fluids.

For this reason, the device must only be operated as follows:

- Only in dry inside rooms (e.g. never in the bathtub or sauna).
- Only with dry hands.

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet. Do not use the device if damage to the device itself or any of its accessories is evident. Do not subject the device to any impact and do not drop it.

Repairs

WARNING

- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorized dealer.
- The zip fastener on the Shiatsu massage device must not be opened. It is only fitted for reasons relating to the manufacturing of the device.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

WARNING: Fire hazard

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.

Handling

CAUTION

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

- Do not insert any objects in the device apertures or in the moving parts. Ensure that the moving parts can move freely at all times.
- Operate the device in an upright position only, never place it flat on the floor.
- Never sit or stand on the moving parts of the device with all your weight and do not place objects on the device.
- Keep the device away from high temperatures.

Disposal

CAUTION

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

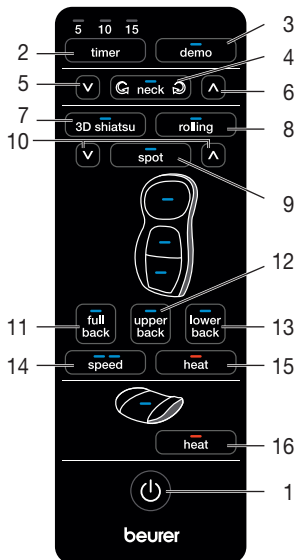
5. Appliance description

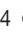
Shiatsu massage pillow (front)

- 1 2 neck massage fingers
- 2 4 massage heads
- 3 Fastening straps
- 4 Mains cable
- 5 Storage bag
- 6 Hand switch
- 7 Removable and washable covers
- 8 Coat hanger loops (rear)
- 9 In-seat heating

Hand switch

- 1 Button 
- 2 **timer** button (5–10–15 minutes)
- 3 **demo** button
- 4  **neck**  button
Neck massage activation in clockwise and anti-clockwise direction
- 5 Height adjustment button  (neck massage)
- 6 Height adjustment button  (neck massage)
- 7 **3D shiatsu** button (shiatsu massage)
- 8 **rolling** button (rolling massage)
- 9 **spot** buttons (localised massage)
- 10 Height adjustment button (localised massage)
- 11 **full back** button (entire back massage)
- 12 **upper back** button (upper back massage)
- 13 **lower back** button (lower back massage)
- 14 **speed** button (massage intensity)
- 15 **heat** button (heating)
- 16 **heat** button (in-seat heating)

6. Start-up

CAUTION

- **IMPORTANT: Before using the device for the first time, ensure you have released all transport locks on the massage mechanism. To do this, remove the screws using the tool provided on the rear of the seat cover. If the transport lock screws are not removed first and the Shiatsu seat cover is used immediately, the device will become irreparably damaged and unusable. The warranty will not apply in this instance.**
- Reuse the transport lock screws when transporting the device in the future. Transporting the device without using the transport locks could cause irreparable damage to the device.
- Position the seat cover upright on a suitable seat with a sitting area and backrest (chair, sofa or similar). Make sure that the seat has a sufficiently high backrest.
- Use the flexible fastening straps and hook-and-loop fasteners to secure the Shiatsu seat cover.
- Connect the device to the mains by plugging the plug into the socket.

7. Operation

Gently press your back against the massage device to start with. Then make sure that you are in a comfortable position before gradually shifting your weight onto the massage device. The massage must be pleasant and relaxing at all times. Stop the massage or change your position or the contact pressure if you find it is painful or unpleasant.

- Do not force or jam any part of your body, in particular your fingers, between the moving massage elements or their holders in the device.
- Sit comfortably and upright on the seat cover. Ensure you sit centered and such that the massage heads can move to the left and right of your spinal column.
- Switch on the massager with the  button. The LEDs signal the massage functions which are currently selected.
- The automatic switch-off function is activated with selection of the massage region and is preset to 15 minutes. You can set the automatic switch-off function to 5, 10 or 15 minutes by pressing the **timer** button. Please note that changing the timer setting makes the timer start to count again from the beginning. To avoid overheating the device, select the maximum massage duration at the beginning of the massage and do not change it again during the massage.

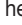
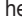
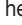
You can specify the settings as follows:

1. Demo run

Press the **demo** button to begin a quick 5-minute run-through of the functions contained in the device. The manual controller indicates which function is being displayed by flashing the respective LED. The device switches off automatically once the demo run has finished.

2. Shiatsu neck massage

Press the  **neck**  button to start the neck massage. Press the  **neck**  button again to change the massage direction. Press the  **neck**  button again to switch off the neck massage.

You can adjust the height of the neck massage by holding down one of the two arrows   next to the  **neck**  button until the desired height is reached.

3. Shiatsu back massage

Press the **3D shiatsu** button. Select the desired massage region with **full back**, **upper back**, **lower back**.

You can choose a **spot massage** i.e. a Shiatsu massage without vertical movement by pressing the **spot** button. You can change the position of the spot massage by holding down one of the two arrows   next to the **spot** button until the desired position is reached.

Massage intensity

When using the Shiatsu Massage, two different massage speeds are available, which can be selected using the **speed** button. If you have the rolling massage activated, first press the **3D shiatsu** button in order to change to Shiatsu Massage and then set the desired massage speed.

4. Rolling massage

Press the **rolling** button. Select the desired massage region with **full back, upper back, lower back**.

When the massage type (Shiatsu or rolling massage) is changed, the preselected region (full, upper, lower back) is retained. Note that the massage region must be selected again when switching from localised Shiatsu massage to rolling massage.

5. Heating function

Press the **heat** [15] button to switch on the heating function of the massage heads. The blue lights in the massage heads change colour to red.

The device features an in-seat heat function. This can be activated using the **heat** [16] button.

- Do not use the massage device for longer than 15 minutes and then allow it to cool. Using the device for longer can over-stimulate the muscles and cause tension instead of relaxation. The massage can be finished at any time by pressing the  button. Please note that on switching off, the massage heads first return to the starting position at the lumbar region. This is indicated by the LED flashing. Once this position is reached, the device switches off.
- If the device is stopped by pressing a massage area button twice, it also returns to the starting position after 15 minutes and then switches off.
- Do not disconnect the device from the mains supply during operation. Always switch it off first and wait until it has returned to the initial position.

Note

- The device has an automatic shut-off mechanism set to a maximum operating period of 15 minutes.

8. Care and storage

Cleaning

WARNING

- Disconnect the device before cleaning it.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary. Do not use any cleaning agents containing solvents.
- The removable neck cover can be machine washed at 40°C. Please note the symbols for care and cleaning on the label sewn into the cover.
The PU cover at the back is secured with a hook-and-loop fastener under the material bearing the beurer logo and can be removed from there.
The cover in the neck area can be removed with the zip fastener.
- Do not use the device again until it is completely dry.

Storage

If you do not use the device for longer periods, we recommend storing it in the original packaging in a dry place without weighting it down.

Alternatively, you can hang the device in a wardrobe with a coat hanger. To do so, the coat hanger must be threaded through the hanging loops on the rear side of the device.

Ensure that you use a stable clothes hanger made from wood that is able to carry the weight of the device.

CAUTION

- If you use the transport lock screws, take care to remove these before starting the device when you next use it.

9. In the event of problems

Problem	Cause	Remedy
Massage heads are moving slowly.	The load on the massage heads is too great.	Reduce the contact pressure on the head.
Massage heads are moving in the upper or lower area only.	The massage has been activated for the limited region.	Press the full back , upper back , lower back buttons to change the massage region. Press the neck button to activate the neck massage.
Massage heads not moving.	Appliance not connected to the power supply.	Connect the plug to the mains supply and switch on the appliance.
	Device not switched on.	Switch the device on with  . Select the massage region.
	Transport locks were not removed. Massage mechanism is damaged.	Contact your authorized dealer or the service address.
	Overheating protection tripped, device is damaged.	Contact your authorized dealer or the service address.

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